

PUSH RIMS INSTRUCTIONS



DO IT YOURSELF
wheelchair

To make the push rims, only need a plastic (PVC) flexible drainage pipe. Cut it lengthwise, and put it in the pneumatic tire before inflating the wheel.

MATERIAL

- 4 meters of Plastic (PVC) flexible drainage pipe, 40 cm diameter.
- Knife
- Inflator bike

For each wheel, we need about two meters of pipe. First, cut the tube, presenting over the wheel.



Cut the pipe lengthwise with a knife or scissors. As straight as possible



With the pneumatic deflated, place the tube between the pneumatic and tire, taking advantage of cut.



Inflate the wheel holding the two ends of the pipe.

