

ARMREST INSTRUCTIONS



DO IT YOURSELF
wheelchair

To make the armrest only you only have to buy more and pipe and tees. Change of materials respect structure without armrest:

MATERIAL CHANGED

- 6.5m Pressure PVC pipe (40 mm diameter)
- 10 elbows (40 mm diameter)
- 12 tees (40 mm diameter)
- 2 seelves (40 mm diameter)

Now cut the pipe following this table, and mount a structure like this, (without glue for the moment). Start by cutting long tubes, trying to take advantage of the tube.

LENGT (CM)	UNITS
41.5	10
31	2
19	4
16	4
6	10

